

left click holding

A simple tap of the button activates the function of pressing and hold-down the left click button, which enables you to select and/or drag windows, widgets, files and other objects. The Led flash and an audible warning will indicate that this function is activated. To unlock it, you must press the button again, then the Led and the audible warning will shut off.



speed selector

To change the cursor movement speed, follow these instructions :

- 1 Press the "Speed Selector" button. A beep will sound indicating the access to this function.
- 2 The blue Led will turn on with a proportional frequency to the actual speed.
- 3 Using the Up/Down movement buttons, it will be possible to modify the cursor movement speed. The Led will flash and a beep will sound according the speed change.
- 4 Once the speed has been selected, you must press the "Speed Selector" button to validate your selection.



assistance request

The AdMouse CSMX1 allows the user to request assistance by pressing a simple button. On executing this function, a beep will sound. It is possible to Activate/Deactivate this function by pressing the "Assistance Request" button for 15 seconds. A beep will sound for this action.



Limited Warranty

Thank you for your interest in our product.

This Limited Warranty applies to the ADMouse purchased from BRANCO INC.

What does this limited warranty cover?

This Limited Warranty covers any defects in material or workmanship under normal use during the Warranty Period. During the Warranty Period, BRANCO will replace, at no charge, the device that proves defective because of improper material or workmanship, under normal use and maintenance.

What will we do to correct problems?

BRANCO will replace the device, with a new or refurbished one.

How long does the coverage last?

The Warranty Period for your ADMouse purchased from BRANCO is one (1) year from the date of purchase. You must register for your warranty to be active.

What does this limited warranty not cover?

This Limited Warranty does not cover any problem that is caused by:

- Conditions, malfunctions or damage not resulting from defects in material or workmanship

What do you have to do?

To obtain warranty service, you must first

Register your device

Name
Address
Contact Phone Number
Email
Serial Number

Email this information to Sales@brancoinc.com

User Guide

v2.2

CSMX1
AdMouse USA
Accessible pointing device

BRANCO INC
1348 NW 113 Terrace
Coral Springs, FL 33071
T:954.253.1050



For children and adults who suffer from involuntary movements, tremors, upper limb paralysis or attentional difficulties.

Allows people with Parkinson's disease and tic disorder, among others, an individual use of a computer.

Help elderly people to access information technologies.

Introduction

The Ad Mouse GSMX1 was built and designed for individuals with varying degrees of motor control difficulties, which often renders them unable to use traditional mouses.

Our button-driven peripheral removes the inherent barrier with conventional mouse functions (cursor movement, click, double click, right click, among others)

Please read this manual thoroughly before plugging in or installing this device into your computer.



Installation and turning on



This device should be placed on a flat and stable surface when in use. We do not recommend using it outdoors or in potentially unsafe and damaging environments. Avoid exposing it to water and moisture. To use it, simply connect it to an available USB port (it requires no batteries and is self-powered)

automatically find and install the appropriate drivers for you. No additional drivers or software are necessary to operate the device. Once the device is ready to use, the red Led light next to the power symbol will light-up.

USB 2.0



NOTE: Holding down the "Speed Selection" button for 5 seconds will activate / deactivate the audible warning of the function keys.



usage

The device has 10 buttons that allow the user to perform the same functions as a traditional mouse, plus some additional features. These buttons are specially designed to allow a simple and intuitive keyboard management, avoiding unintentional keystrokes. The buttons are in low relief, allowing a comfortable one-touch.



4 directional buttons - horizontal and vertical movement. Move the cursor by pressing one of the red buttons with directional arrows.




Left Click - Easy to use by its central position.



Right Click (Secondary Click) - Green button with a white square in its center. Shows additional functions.

command distribution

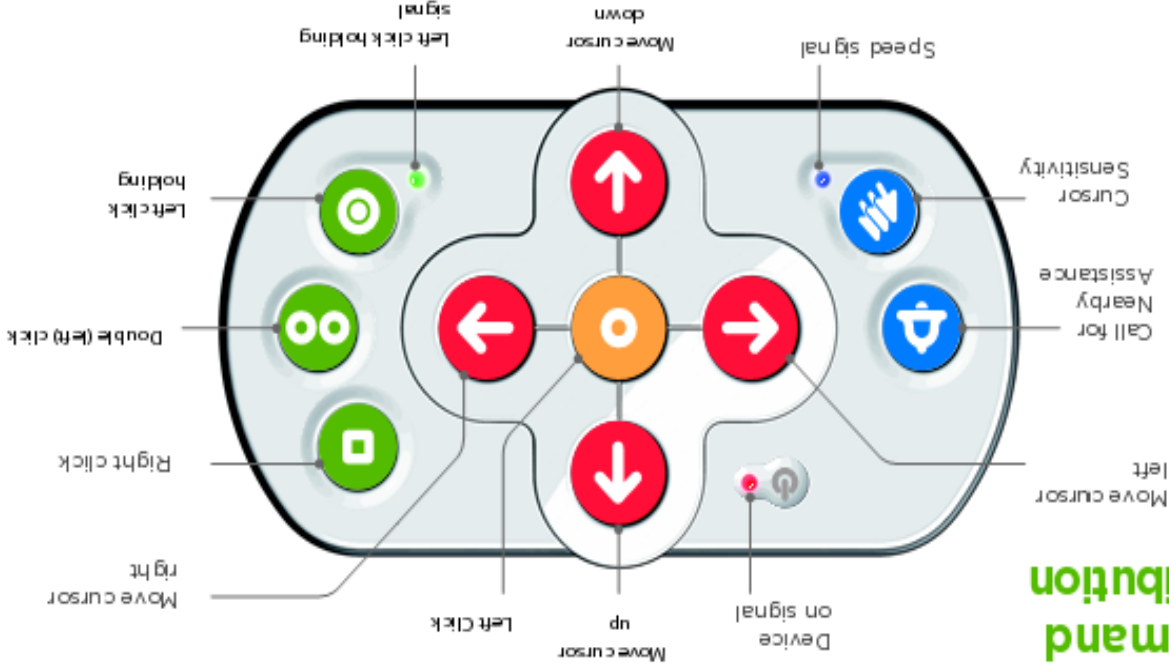

Double (left) Click Eliminates the need of making two clicks to the Left Click Button.



Left Click Holding When pressed, allow to select and move an icon or other object.



Speed Selector Increase (+) or Decrease (-) mouse cursor speed.



Assistance Request Beeps to call attention of your assistance.